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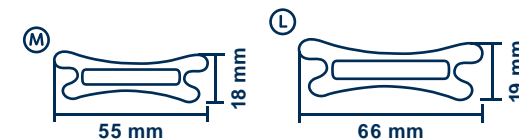
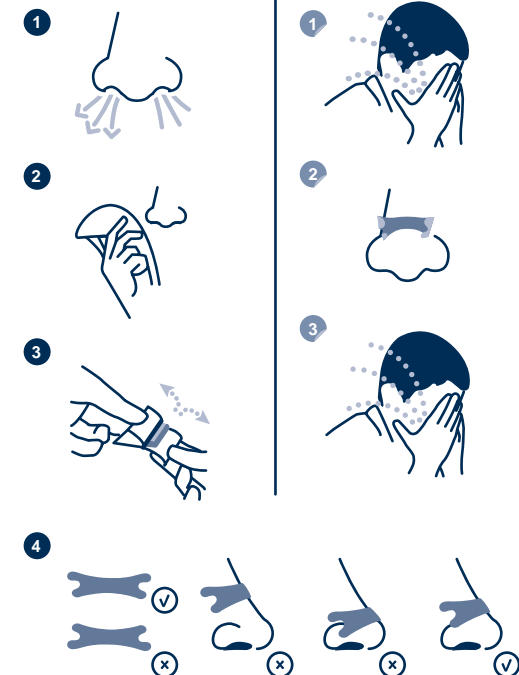
Removal / Entfernung / Rimozione / Comment retirer la bandelette / Remoción

Manual • ENGLISH

oxyhero pull Nasal strips are applied on the outside of the nose. They are made of adhesive and flexible material, such that no further auxiliary means are necessary to affix them. oxyhero pull widens the nasal passages and therefore provides for an increased air intake through the nose. Not suitable for children under the age of 6 years.

Area of application:

- Providing relief from allergies and common colds by creating a greater supply of air.
- Reducing the frequency and intensity of snoring by minimising phases of oral respiration.
- Increasing air supply when doing sports.

Safety information:

- oxyhero pull nasal strips are solely intended for external use.
- Each strip can only be used once and has to be disposed of afterwards.
- Only apply oxyhero pull nasal strips on healthy skin, which should be cleaned and dried beforehand.
- Do not affix nasal strips if you have a sun burn or suffer from skin irritation, eczema or the like.
- oxyhero pull nasal strips should not be worn on the skin for more than 12 hours.
- Should skin irritations or allergic reactions occur, discontinue use of oxyhero pull nasal strips immediately.
- Store at room temperature and keep out of reach of children.
- Not suitable for children under the age of 6 years.

How to use:

Affix oxyhero pull 10 to 30 minutes prior to desired onset of effect.

Application:

- 1 First clean the nasal passages if necessary. For that purpose you can make use of nasal irrigation.
- 2 Clean and dry the outer skin of your nose.
- 3 Remove nasal strips from their secondary packaging.
- 4 Affix the oxyhero pull nasal strip on the middle nose bridge and lightly press the adhesive surface with your fingers onto your nasal wings (also take a look at the graphic printed on the nasal plaster packaging).

Removal:

- 1 Wash your face with warm water in order to gently loosen the oxyhero pull nasal strip from you skin.
- 2 Slowly remove both ends of the plaster from your nasal wings after which you can take off the fixation on the nasal bridge.
- 3 After removal, thoroughly clean the skin of your nose again such that any adhesive residues that still remain are completely washed away and skin irritations are prevented.

Side effects:

Skin irritations and allergic skin reactions at the point of application

Important note:

Nasal strips are not intended for treating sleep apnoea. In the case of suspected sleep apnoea, you should in any event consult a doctor first.

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